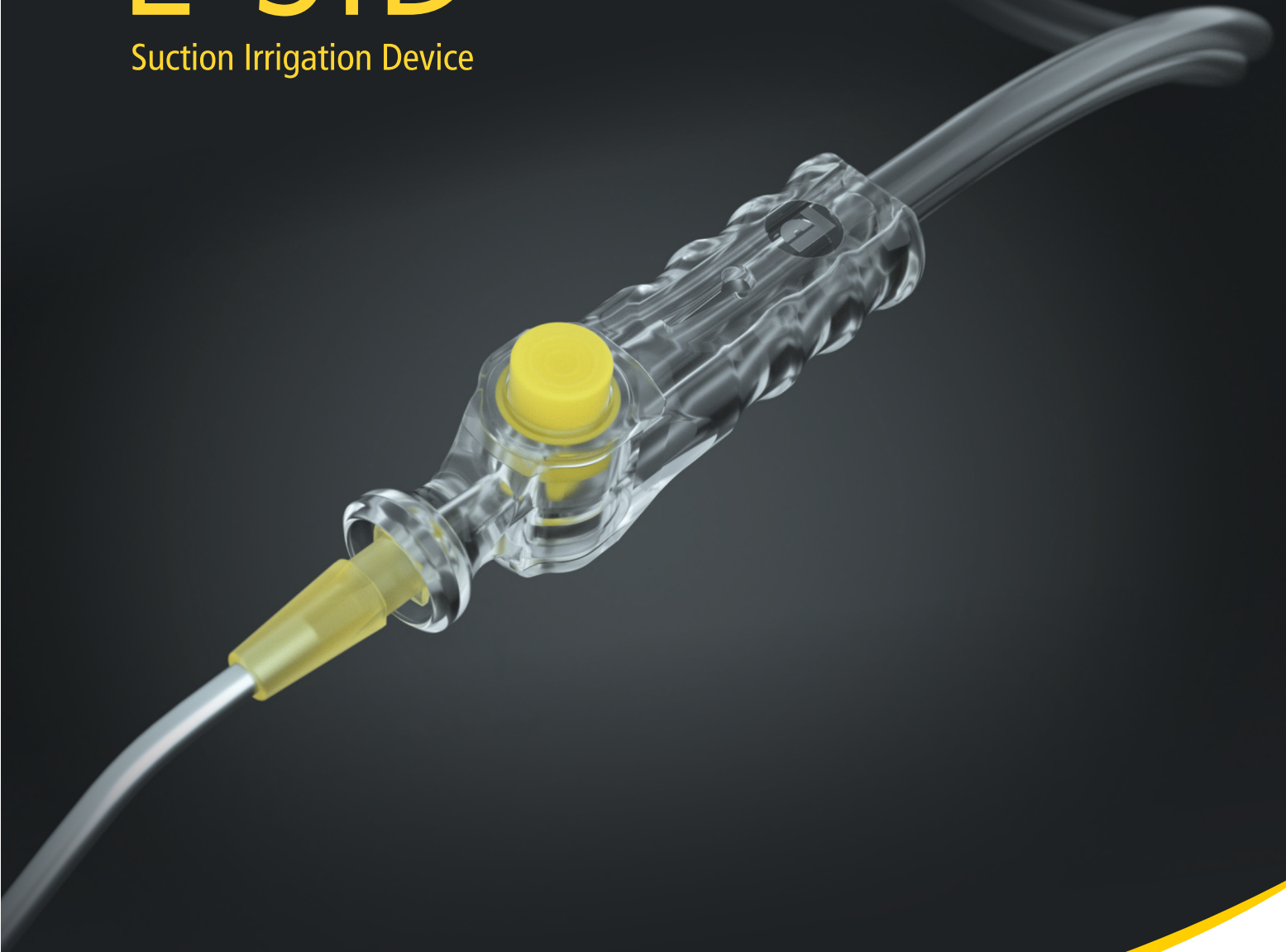
L-SID

Suction Irrigation Device

lazic.de

PETER LAZIC
MICROSURGICAL
INNOVATIONS

L-SID OUR DISPOSABLE SUCTION-IRRIGATION DEVICE FOR MICROSURGERY.

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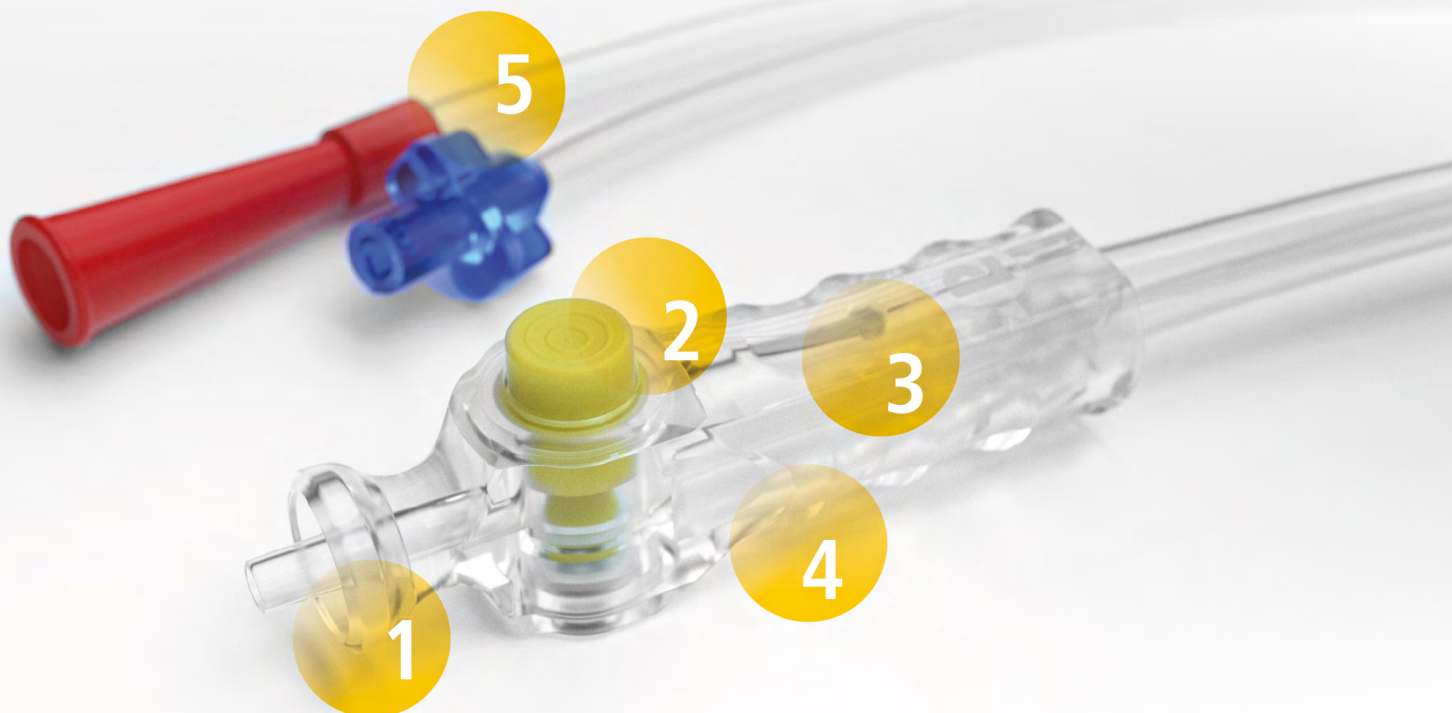
Suction Irrigation L-SID optimizes workflow and efficiency in the OR. The ergonomic, transparent handle design and ease of use allow fatigue-free work even during longer microsurgical procedures. Without the help of additional OR staff, microsurgeons can perform the irrigation procedure themselves when using L-SID Suction Irrigation, thus enabling a better view to the surgical field.

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LUER-LOCK CONNECTOR

Luer-Lock-Anschluss

Various cannulas adaptable up to an inner diameter of 3 mm. Cannula can be rotated 360° as desired.

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IRRIGATION BUTTON

Spülknopf

For switching from suction to irrigation with two in one function: irrigation and suction with one single device.

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TEARDROP

Teardrop

For regulation of suction power. The teardrop opening allows optimal metering of air supply and suction power.

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ERGONOMIC HANDLE DESIGN

Ergonomisches Griffdesign

For fatigue-free work - even during extended use. Intuitive and easy operation of irrigation button and suction opening. Fast, on-demand switching from irrigation to suction and vice versa.

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COLOR CODED CONNECTORS

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To identify suction pump and irrigation device. With integrated sterile suction- and irrigation tube (3m) for easy and safe connection.

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SUCTION POWER REGULATION (TEARDROP)

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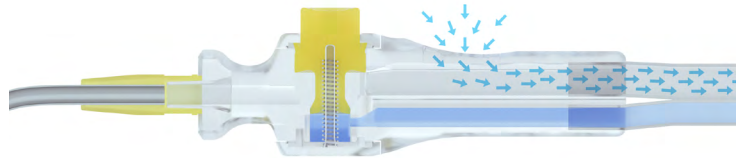
Without active intervention, Suction Irrigation L-SID is in open circuit condition. The suction power can be regulated continuously by the amount of inflowing air at the teardrop opening.

- suction line is inactive
- irrigation line is closed

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FUNCTIONAL PRINCIPLE SUCTION

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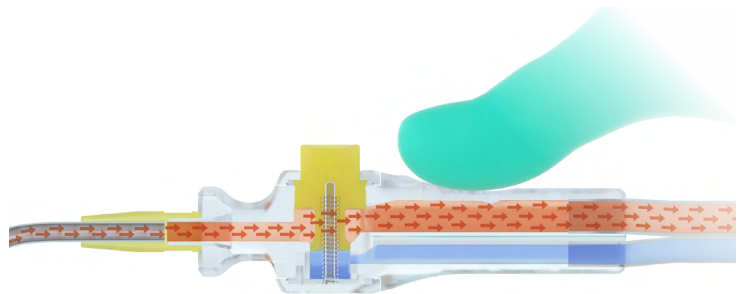
by closing the suction regulation (Teardrop)

- activation of suction line
- irrigation line is closed

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FUNCTIONAL PRINCIPLE IRRIGATION

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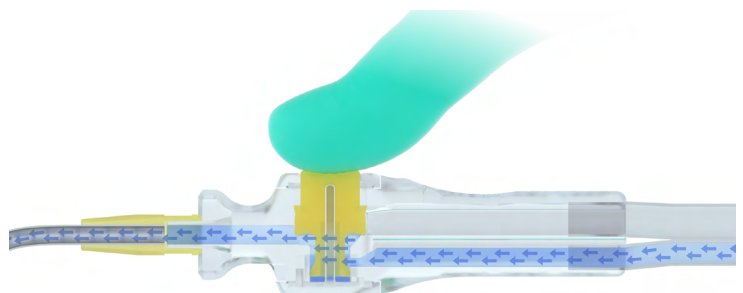
by using the irrigation button

- activation of irrigation line
- deactivation of suction line

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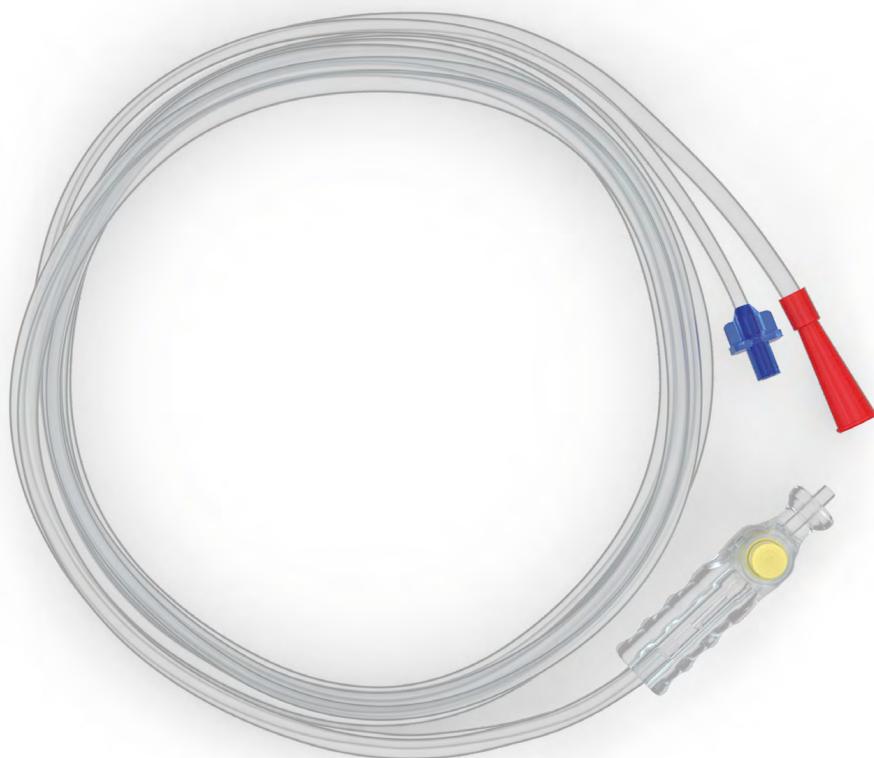
latex free

76.100.12

**Suction Irrigation Device sterile
with PVC-tube (3m), PU = 12 pcs.**

Spülsauger, steril mit PVC-Schlauch (3 m),
VE = 12 Stk.

ACCESSORIES

Zubehör

Further accessories available on request:

- Cannulas (standard Luer-Lock)
in various lengths and diameters
- Pressure infusion cuff

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SINGLE-USE CANNULAS

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Can be combined with all cannulas with standard Luer-Lock connector up to 3 mm inner diameter. Cannulas in various lengths and diameters on request.

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Tube/Rohr ø 3,5 mm, 160 mm

19.102.02

**Storage tray for 9 tubes,
incl. lid, with inlays,
267 x 122 x 55 mm**

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CE 0297

All rights reserved. Reprinting, also in excerpts, is forbidden. No part of the catalogue may be reproduced in any form or processed, duplicated or distributed using electronic systems without the written permission from PETER LAZIC GMBH.

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Peter Lazic GmbH
Peter Lazic US Inc.